

User Manual



Before using, please read the user manual carefully and follow the instructions to operate properly.



Model No.: CAD0001


Thank You!

Thank you for purchasing the Clean-Z CPAP Sanitizer, which is used to sanitize germs and bacteria by producing O₃. You can use it to sanitize your CPAP mask and tubing. The entire sanitization requires no water or other chemical cleaning.

Warning:

- *Keep this product out of the reach of children.
- *Do not disassemble or attempt to repair this product, as it may cause damage to the device.
- *Do not get the device wet and avoid all liquids when using it.
- *Do not use in combustive air environments, near gas vapors or other flammables, etc.
- *Use the original power adapter and USB cable.

Product Description

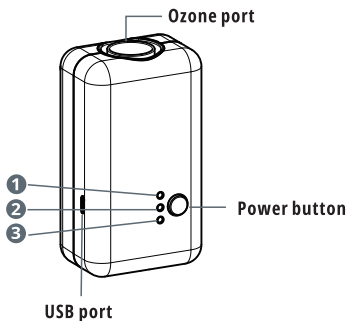

1. Product parts



Ozone Cleaner




Power Adapter



Tubing Adapter



USB Cable

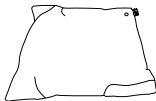

Sanitizing Bag

2. Operation Instruction

1) Start-Up Stage: When the Clean-Z is powered on there is a one minute preparation period with a single beep. You will see a slow flashing red LED ① light with no ozone produced.



2) Cleaning Stage: After the start-up stage, the Clean-Z will start to produce ozone and work continuously for 30 minutes. The red LED light will cycle from LED ① to ③ .



3) Fresh Air Stage: The Clean-Z will stop producing ozone with a green LED light cycling from LED ① to ③ . The fan will run for five minutes to flush the residual ozone.




4) Cleaning Complete: The Clean-Z will stop working automatically with three green LED lights on and three beeps.




Application

To clean the CPAP mask and tubing:

Put the mask and tubing into the sanitizing bag and connect the Clean-Z by the ozone port. Power the Clean-Z on to start the sanitization process.

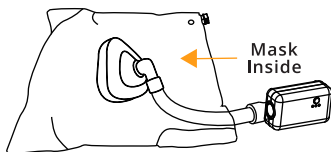

Filter Cartridge: This filter converts ozone back into regular air. Replace the filter once every six months.

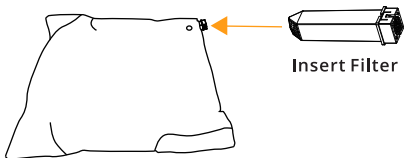

Warning: After the sanitization process, wait at least an hour before using the CPAP equipment to ensure no ozone resides.

Specifications

Size	4.1x2.4x1.6 in
Weight	4.4 ounces
Air flow	3.5 CFM
Powerconsumption	10W
Power suply	DC 4.5V~6.5V
Store temperature	-4 to 131 °F
Operation temperature	23 to 113 °F

Routine Care

1. When not in use, please store your Clean-Z.
2. Do not place the Clean-Z in a damp area or in direct sunlight for a prolonged period.
3. When using the Clean-Z to sanitize, please put it in a dry, clean and ventilated environment.
4. Keep the Clean-Z and the sanitizing bag dust-free by periodically wiping it down with a damp cloth. If the bag gets dirty, you may hand wash it and allow it to air dry. Do not machine wash or machine dry the bag.

Troubleshooting

Problem	Possible Solutions
The Clean-Z does not work after turning on.	Connect the Clean-Z to the adapter. Plug the adapter into the power source. Turn on the Clean-Z.
No flashing light or beeping sound	Make sure the power connection is correct. If the problem is still not resolved, please send it in for repairs.
The Clean-Z doesn't stop automatically. .	Turn the device off and then on. If the problem persists, contact us at (844) 272.8390.
No 03 generation in one minute after it is powered on.	Power the Clean-Z on again. If the problem persists, contact us at (844) 272.8390.



This marking indicates that this product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle this product responsibly to promote the sustainable reuse of material resources.

To return your used device, please use the return and collection system or contact the retailer where the product was purchased. They can safely recycle this product.

**Manufactured for:
Motif Medical, LLC
Dover, DE 19901
www.motifclean-z.com
(844) 272.8390**